


JET RING

11 ACB

Atraumatic **C**lockwise
Preloaded capsular tension ring

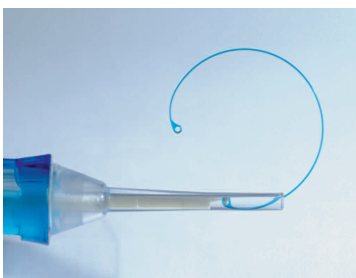



1. Gently remove the injector from its packaging box
2. Pull the security holder backwards in order to load the ring into the injector
3. Laterally remove the security holder
4. Push on the piston to inject the ring

Technical data

Injector	Preloaded
Ring Overall Diameter	11 mm
Material of the ring	PMMA blue
Type of lens	Atraumatic
Sterilization	Ethylenoxid

Main features & benefits

- Preloaded
- Single use
- Ready to use and time saving

Indications

- Cataract surgeries of subluxated lenses
- Zonular weakness and desinsertion
- High myopia
- Prevention of capsular bag shrinkage in patients with congenital cataract