

Disposable Injection Kit

MEDJET 2.2




MEDI  **NTUR**


Vision of Expertise

MEDJET 2.2


Disposable Injection Kit

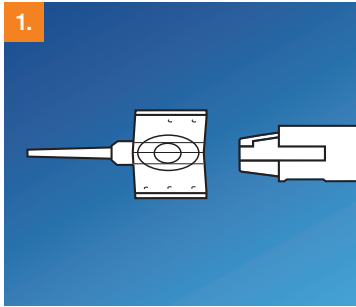

1. Lubricate the funnel and the loading barrel of the cartridge with viscoelastic solution.

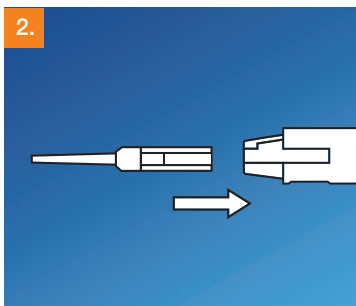

2. Place the lens in the middle of the cartridge. Carefully push the optic and the haptics under the guiding rail and close the cartridge.

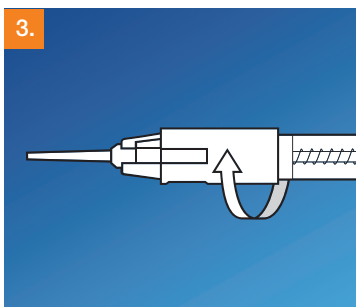

3. Fill up the cartridge with viscoelastic solution behind the lens.

4. Load the cartridge and lock it in place by turning the ring clockwise.

5. Gently inject the lens while rotating the injector from 3 o'clock to 12 o'clock.



Technical data

| | |
|------------------------|--|
| Pusher | Silicon cushion for hydraulic injections |
| Loading chamber | with ring locking system |
| Type of lens | One-piece |
| Sterilization | Ethylene Oxide |
| Packaging | Special blister "2 in 1": Cartridge + Injector |

Cartridge

| | |
|--------------------------|--|
| Winglets | with special locking system |
| Loading chamber | with guiding rails for placing the lens easily |
| Internal diameter | 1.30mm |
| External diameter | 1.58mm |
| Incision size | 2.20mm |