

Recommendation List: ologen™ collagen matrix

ologen™ collagen matrix is a product for eye tissue repair. It prevents ophthalmic surgical fibrosis by inducing the fibroblast to grow randomly in the mimic extra cellular matrix. By this way, the natural and physiological tissue is established.

This recommendation is just as a reference and surgeon should decide a right size and approach himself/herself.

Intervention	Recommendation (model number)
Trabeculectomy	830601, 830621, 862051 <i>Please be reminded that loose suture over the scleral flap is very important. Too tight will make the surgery fail.</i>
Strabismus surgery	870051, 870061, 830661
Pterygium excision	870051, 870061, 830661, 830681
Combine with shunt/valve	830601, 830621, 862051
Subconjunctival scar revision	830601, 830621, 862051, 830661, 830681
Corneal ulcer	862051
Eye plastic surgery	Please check the product size and decide an appropriate one.
Retinal surgery	
Nonpenetration deep sclerectomy	870051 <i>You may cut a small piece for the scleral tunnel and big piece on top of the scleral flap. Loose suture is strongly recommended</i>

Note. The information of model number and its dimension is list on Page 2

For further technical information, please contact **Dr. Julio Lin** at Julio.lin@aeonastron.net

Reference

Model Number	Dimension
830601	6 mm (D) × 2 mm (H)
830621*	7.5 mm (D) × 2 mm (H)
830661	7 mm (D) × 4 mm (H)
830681	7.5 mm (D) × 4.5 mm (H)
862051	12 mm (D) × 1 mm (H)
861051	12/20 mm (ID/OD) × 1 mm (H)
870041	5 mm (W) × 10 mm (L) × 4 mm (H)
870051	10 mm (W) × 10 mm (L) × 2 mm (H)
870061	10 mm (W) × 10 mm (L) × 4 mm (H)

Important Note:

Model Number: **830621*** will be available from April 2010.

For further product information, please contact us at info@aeonastron.net

Aeon Astron Europe B.V.

Niels Bohrweg 11-13, 2333 CA Leiden, The Netherlands

Tel: +31 71 3322280 Fax: +31 71 3322281

Web: www.aeonastron.com or www.ologen.com

2010. Feb